HUMANTECHNIK

lisa

DE

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Seite 2

Funk-Blitzlampe

EN

Operating Instructions

Page 8

RF flash lamp

FR

Mode d'emploi

Page 14

Lampe-flash radio *lisa*

NL

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Pagina 20

Draadloze flitslamp

IT

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Pagina 26


Radiolampada flash

ES

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
Página 32

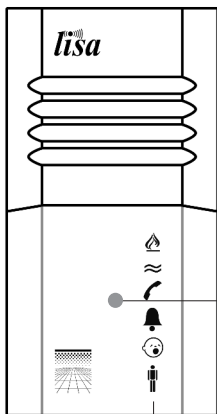Radiolámpara de destellos


Blitz-
fläche

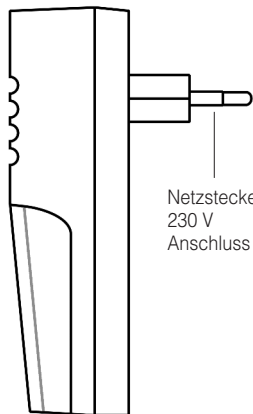Signallämpchen

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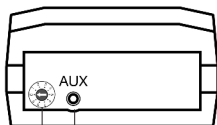
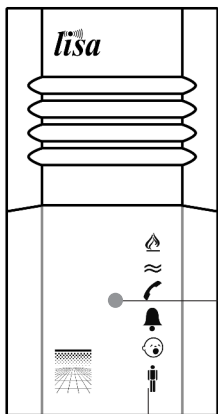
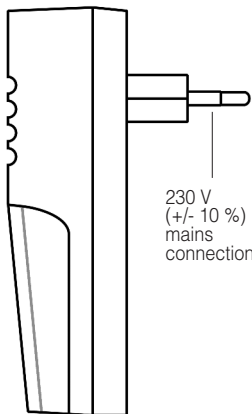
LED displays for
lisa signals

230 V
(+/- 10 %)
mains
connection

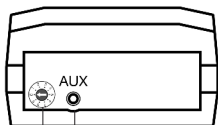Socket for vibrating pad and modules (special accessory)

Channel selector switch

Congratulations on purchasing your cordless *RF flash lamp*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read through the operating instructions carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

Standard components

Please check if all following components are included:

- *RF flash lamp*
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

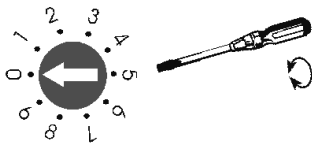
Functional principle

The *RF flash lamp* receives signals from various transmitters and converts them into light signals. The *RF flash lamp* has a powerful flash tube which alerts you to a signal and LEDs displaying the type of signal being received.

Setting up the system

Setting the correct radio channel

A special digital transmission process ensures that the signals are picked up without any errors. This process also enables 10 different radio channels to be set. The default setting for the cordless *RF flash lamp* is channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Using a small screw driver, you can set the channel number you would like to use.

Installing the cordless *RF flash lamp*

The various transmission signals are converted into different sequences of flashes and signal displays by the cordless *RF flash lamp*:

Telephone: Telephone LED lights
flashing sequence

B	P14	B	P30	W
---	-----	---	-----	---

Doorbell I: Doorbell LED lights
flashing sequence

B	P14	B	P14	B	P14	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

Doorbell II: Doorbell LED flashes
flashing sequence

B	P20	B	P20	B	P20	B	P40	W
---	-----	---	-----	---	-----	---	-----	---

Babymonitor: Babymonitor LED lights
flashing sequence

B	P35	B	P40	W
---	-----	---	-----	---

Person call: Person call LED lights
flashing sequence

B	P12	B	P12	W
---	-----	---	-----	---

Alarm: All LEDs flash
flashing sequence

B	P20	B	P20	B	P14	B	P14	B	P14
---	-----	---	-----	---	-----	---	-----	---	-----

Water alarm: The water alarm LED lights
flashing sequence

B	P12	B	P20	B	P12	B	P12	B	P12	W
---	-----	---	-----	---	-----	---	-----	---	-----	---

Fire alarm: Fire alarm LED lights
flashing sequence

B	P12	B	P20	B	P12	B	P20	W
---	-----	---	-----	---	-----	---	-----	---

P12 pause 1,2 seconds

P30 pause 3,0 seconds

P14 pause 1,4 seconds

P35 pause 3,5 seconds

P20 pause 2,0 seconds

P40 pause 4,0 seconds

B flash

W repeat

The indicator LEDs remain on for 40 seconds.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

Maintenance and care

All *lisa* devices are maintenancefree. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Note: remove the *RF flash lamp* from mains socket! Never use spirits, thinners or other organic solvents.

Please do not expose *lisa* devices to direct sunlight for a long time and please protect them from excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

All *lisa* devices are very reliable products. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company, your communal disposal company or your local dealer.

Technical data

Power supply:	230 V, 50 Hz,
Power consumpt.:	approx. 1 W
Frequency:	868,35 MHz
Height::	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	215 g

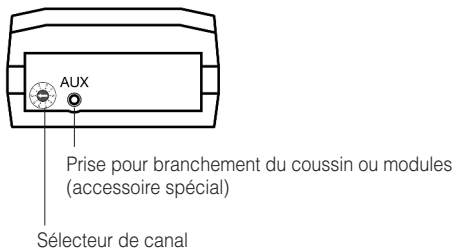
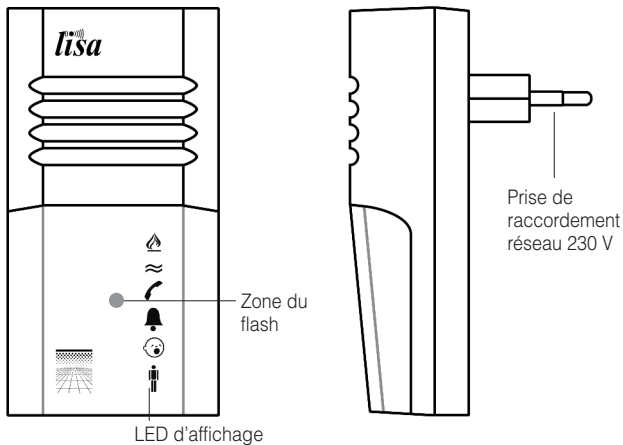
This device satisfies the following EU directives:

- RoHS directive
- WEEE directive
- R&TTE directive
- EMC directive
- Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at http://www.humantechnik.com/downloads/pdf/eu_konf/CE2414-0.pdf

Technical specifications subject to change without notice.

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Sarabec Ltd